

PARTICLE FILTERING HALF MASK

LSD008 FFP2 NR

Following standard: EN 149:2001 +A 1 :2009 Regulation (EU)2016/425

USE INSTRUCTION

1. Failure to follow all instructions and limitations could seriously reduce the effectiveness of this particle filtering half mask and could lead to illness, injury or death.
2. A properly selected respirator is essential, before occupational use, the wearer must be trained by the employer in the correct use of the respirator in accordance with applicable safety and health standards.
3. This particle filtering half mask does not supply the oxygen. Use only in adequately ventilated areas as containing sufficient oxygen to support life.
4. Discard the respirator and replace with a new one if excessive clogging of the particle filtering half mask causes breathing difficulty or the particle filtering half mask becomes damaged.
5. Leave the contaminated area if dizziness, irritation or other distress occurs.

USE LIMITATION



1. Do not use the respirator to enter or stay in a contaminated area under the following circumstances:
 - a) Atmosphere contains less than 19.5% oxygen.
 - b) If you smell or taste contaminant
 - c) For protection against gases or vapors
 - d) Contaminants or their concentrations are unknown or immediately dangerous to life or health.
 - e) For sandblasting, paint-spray operations and asbestos.
 - f) In explosive atmospheres
2. Do not modify or misuse the mask.
3. Do not use the particle filtering half mask with facial hair or any other conditions that may prevent a good face-seal, the requirements for leakage will not be achieved.
4. Particle filtering half mask need to be inspected prior to each use to assure there are no holes in the breathing zone other than punctures around and staples and no damage has occurred. Enlarged holes resulting from ripped or torn filter material around staple punctures are considered damage.
5. This respirator helps protect against certain particulate contaminants but does not eliminate exposure to the risk of contracting disease or infection. Misuse may result in sickness or death.
6. This particle filtering half mask marked "NR", shall not be used for more than one shift.

FITTING INSTRUCTION

- 1) Open the mask and pull the earloop straps with both hands. Ensure that the nose clip is above the mask.
- 2) Cover your mouth and nasal area with the mask.
- 3) Pull the earloop straps behind your ears. Hang the hook to the straps for extension.
- 4) Adjust the hood according to desired comfort.
- 5) Adjust the tightness of the mask according to face shape. Press the nose clip to adjust tightness, exhaling to make sure no air leaks out.



- 6) Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
- 7) Change the mask if a proper face seal cannot be achieved.
- 8) Careful observance of these instructions is an important step in safe respirator use.

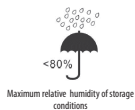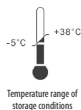

Particulate respirators meet the requirements of Regulation (EU) 2016/425. They have been tested according to EN 149:2001 +A1 :2009 FFP2 NR.

CERTIFICATION BY

CCQS Services Limited

Address: Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15 D15 AKK1, Dublin Ireland

Manufacturer:

Shandong C.I.R.S Garments Co., Ltd.

Address: Mishui Industrial Park Xingfu Road, Gaomi City, Shandong Province, P.R. China

Imported by : PORTWEST - Westport, Co. Mayo, F28 FY88, Ireland. Download the Declaration of conformity : bit.ly/3gtk0XX

EU DECLARATION OF CONFORMITY
Annex IX PPE Regulation (EU) 2016/425

This EU Declaration of conformity refers to the following products

Product Name	Model	Classification/Type	Batch No./Serial No./Identifier
Particle filtering half mask	LSD008	FFP2 NR	LOT NR = 20200514A

The Manufacturer's name and address is as follows:

Name: Shandong C.I.R.S Garments Co., Ltd.

Address: Mishui Industrial Park, Xingfu Road, Gaomi City, Shandong Province, P.R. China

This Declaration of Conformity is issued under the sole responsibility of the Manufacturer.

Detailed description of the PPE to allow traceability/ identification of the PPE.

LSD008

White folded half mask without valve



The article identified in (4) above is in conformance with the relevant Union Harmonization Legislation Regulation (EU) 2016/425.

References to the relevant harmonized standards used, including the date of the standard, or references to the other technical specifications, including the date of the specification, in relation to which conformity is declared:

No.	Harmonized standard name
1	EN 149:2001+A1:2009



CCQS Certification Services Limited. (NB 2834) performed the EU Type Examination (Module B) and issued the Type Examination Certificate Number:

No.	EU Type Examination (Module B) Certificate Number
1	CE-PC-200316-056-01-9A
2	

Product Category:

- ☐ This product is Category II.
- ☐ This product is Category III and is subject to Module C2 internal production control plus supervised product checks at random intervals and is under the surveillance of CCQS Certification Services Limited. (NB 2834)
- ☒ This product is Category III and is subject to Module D Conformity to type based on quality assurance of the production process and is under the surveillance of CCQS Certification Services Limited. (NB 2834)